



## R470 Knee Walker HD Owner Manual



[www.tuffcare.com](http://www.tuffcare.com)

Anaheim, California 92807 USA

### Pre-Caution

- \*Do not operate your Knee Walker until you have carefully read and fully understand this manual.
- \*Do not operate your Knee Walker until it is fully assembled and checked.

**Operation** - The knee Walker HD is designed for easy steering and stopping. Brakes are also featured on knee Walker HD for controlled movement and safety.

- 1, Adjust the height of knee walker as necessary.
- 2, Place injured leg on the center of bench and position towards the front end to cover the full length of bench.
- 3, With injured foot pointing down, the propulsion leg should be maintained as close as possible to the bench.
- 4, While on your toes, start out with small steps with propulsion leg. Concentrate on keeping that leg next to the bench. With practice, you should be able to move forward without deviating to opposite side.
- 5, To use brake, simply squeeze the lever. To lock the brake pull the lever then push the nut down. To release, simply pull the lever again.

Note: The knee walker's brake system is similar to a bicycle and also has a locking feature.

### Assemble Instruction -

- 1, Lever Lock can be adjusted either Right or Left handle use - See Figure (1).
- 2, Leg height adjusts to fit height of user - See Figure (2).
- 3, Tool free, and easy adjustment Right or Left support - See Figure (3).
- 4, Handle adjusts to fit the height of user - See Figure (4).



Figure (1)



Figure (2)



Figure (3)



Figure (4)

### Limited Warranty -

The warranty below has been drafted to comply with Federal Law applicable to products manufactured after July 4, 1975. This warranty is extended only to the original purchaser consumer, or dealer non-consumer who does not buy for resale.

Tuffcare warranty that it's proprietary Freedom Cart Series will be free from defective workmanship and materials for a period of (6) months following date of original purchase.

The foregoing warranty shall not apply to products subjected to negligence, abuse, misuse, improper operation, improper maintenance, improper storage or damages beyond Tuffcare's control.

For warranty service, please contact the authorized dealer from whom you purchased your Tuffcare products or contact us at [www.tuffcare.com](http://www.tuffcare.com).

